

	INDIVIDUAL TOOTH MOVEMENTS	CASE TYPES	TYPICAL TREATMENT LENGTH
Beginner Aligners Only	<ul style="list-style-type: none"> - Canines/bicuspid rotations (<45 deg) - Incisor rotations (25-40 deg) - Posterior rotation (<20 deg) - RELATIVE Extrusion - Intrusion (1 mm per arch) 	<ul style="list-style-type: none"> - Mild crowding/spacing (0-3 mm) - Narrow arches - Mild to moderate deep bite (2-6 mm) - Diastema Closure (1-2 mm) - Mild ortho relapse - Individual anterior crossbite 	6-9 months (<20 Aligners)
Intermediate Aligners with potential need for buttons and elastics	<ul style="list-style-type: none"> - Canine/bicuspid rotations (45-55 deg) - Incisor rotations (40-55 deg) - Posterior rotation (>20 deg) - Pure extrusions (<2.5 mm) - Translations (<3 mm) - Intrusion (1-2 mm per arch) 	<ul style="list-style-type: none"> - Moderate crowding/spacing (3-6 mm) - Class II/III correction up to 3mm - Single or double tooth anterior crossbite - Deep bite correction 	10-14 months (20-30 Aligners)
Expert Aligners with likely need for Sagittal Correction and/or combined with Sectional Fixed Appliances	<ul style="list-style-type: none"> - Canine/bicuspid rotations (>55 deg) - Incisor rotations (>55 deg) - Pure Extrusions (>2.5 mm) - Translations (>3mm) - Intrusion (>2 mm per arch) 	<ul style="list-style-type: none"> - Severe crowding/spacing (>6 mm) - Extrusion of high canines, in closure of anterior/posterior open bites (>2.5 mm) - Bicuspid extractions - Posterior crossbite correction - Class II/III correction >3 mm - Surgical treatments - Severe deep bite (>6 mm) 	15-24 months (>30 Aligners)

These guidelines are intended as a general tool only. Treatment decisions and determination of patient suitability for Invisalign treatment remain the responsibility of the treating doctor. Individual tooth movements are unpredictable and cannot be guaranteed.